


Archive data saved in the oximeter using SpO2 Assistant.


1. Connect the oximeter to the computer either via the USB port.
2. Start SpO2 Assostant software.



3. Select Connect Device button (the one that looks like a DB9 socket). A window would show the devices available (see diagram below).

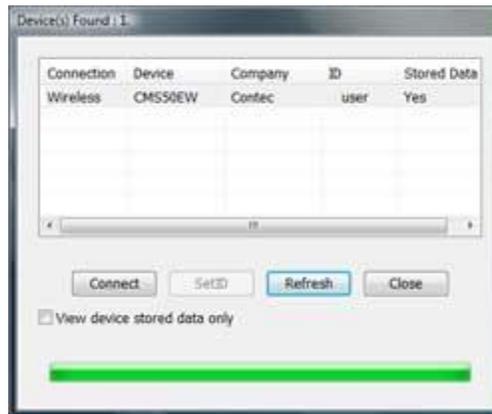

4. Check the "View device stored data only" box for downloading the stored data inside the oximeter. Then click the "Connect" button.
5. The next window would show all the data saved inside the oximeter. Select the one you are interested in and then click the "Receive Data" button.

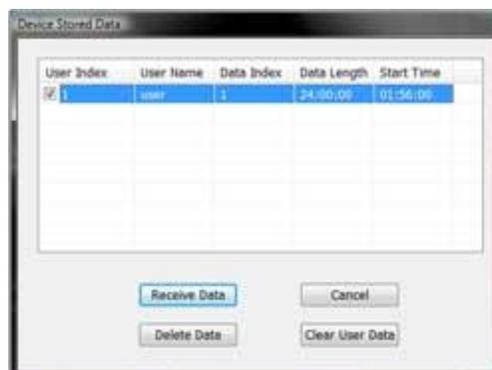

6. The data download would start.



7. To examine the file, select "File" >> "Open File" to open the file you are interested in.

